



Techform® Ortho-Roll® with QUICKDRY Padding


APPLICATION INSTRUCTIONS

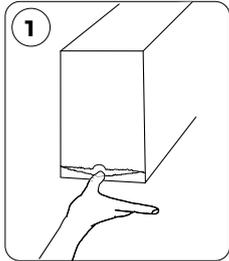

1 Open box to expose end of splint.

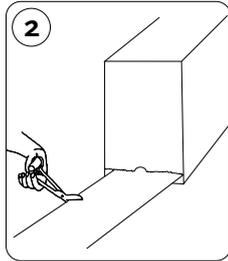

2 Pull out desired length of product and cut.

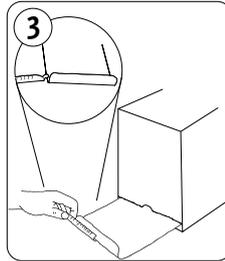


3 Fold the end of splint up inside pouch. Ensuring there are no wrinkles, fold the end of the pouch upwards and place the yellow tip of the open end of the closure rod into the crease at one edge of the pouch fold. Slide the rod onto the pouch (entire width) to ensure an air tight fit.



4 Immerse in clean, room temperature water until saturated.




5 Squeeze out excess water.

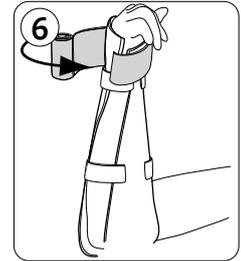

6 Apply splint to extremity. Secure splint as usual with a bandage or similar means.

Product should be stored at room temperature

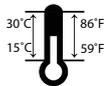

See Instructions for Use



Life Without Limitations®

WARNINGS

Techform Ortho-Roll is intended for external immobilization.

Review all instructions and cautions before using this product.

**SPLINT IS PADDED
ON ONE SIDE ONLY.**

**PLACE PADDING
NEXT TO SKIN.**

1. Use this product only under the guidance of a physician or certified medical professional. If any problems occur with the use of this product, please immediately contact your medical professional.
2. If any discomfort, numbness, or excess heat is felt by the patient, remove splint immediately and bring to the attention of a medical professional.
3. Instruct patient to periodically check circulation of immobilized anatomy.
4. The use of gloves is optional. **Do not allow any of the inner splinting material to come in contact with skin.** The polyurethane resin of the inner splinted material can adhere to unprotected skin or clothing. Swabbing lightly with alcohol may help remove resin if done immediately.
5. To avoid the possibility of thermal burns, **always place the padded side next to skin** and do not rest newly splinted extremity on pillow.
6. Do not apply splint directly over open wounds or other skin conditions which may result in infection.
7. Door must remain closed and latched at all times when not in use to prevent premature hardening of the splint in the package. Care should be taken to avoid puncturing or tearing the pouch in storage, as this may cause premature hardening in the package. Product should be stored at room temperature.
8. Cut fiberglass edges can be sharp. Ensure the edges have been properly trimmed or folded back to prevent skin irritation or injury.

Össur Head Office

Grjóthals 5
110 Reykjavik, Iceland
Tel: + 354 515 1300
Fax: + 354 515 1366
Mail@ossur.com

Össur Americas

27015 Towne Centre Drive
Foothill Ranch, CA 92610, USA
Tel: +800 233 6263
Fax: +800 831 3160
ossurusa@ossur.com

Össur Canada

120-11231 Dyke Road
Richmond, BC
V7A 0A1, Canada
Tel: +1 604 241 8152
Fax: +1 604 241 8153

Össur Europe

Ekkersrijt 4106-4114
P.O. Box 120
5690 AC Son en Breugel
The Netherlands
Tel: +800 3539 3668
Tel: +31 499 462840
Fax: +31 499 462841
info-europe@ossur.com

Össur Iberia

Parque Europolis
Calle Edimburgo n°14
28232 Las Rozas (Madrid) Spain
Tel: +34 91 636 06 93
Fax: +34 91 637 21 43
ortetica@ossur.com

Össur UK

Building 3000
Manchester Business Park
Aviator Way
Manchester M22 5TG, UK
Tel: +44 161 490 8500
Fax: +44 161 490 8501
ossuruk@ossur.com

Össur Asia

上海虹梅路1801号W16B 棟2楼
邮政编号 : 200233
电话 : +86 21 6127 1700
传真 : +86 21 6127 1799
asia@ossur.com



Life Without Limitations®