

Instructions for use

Icelock Smooth 651

Life Without Limitations®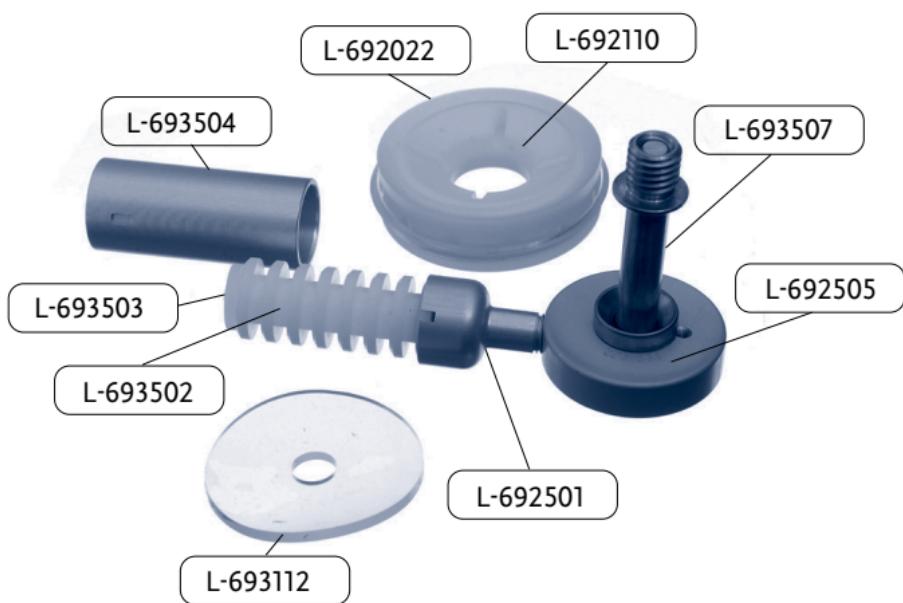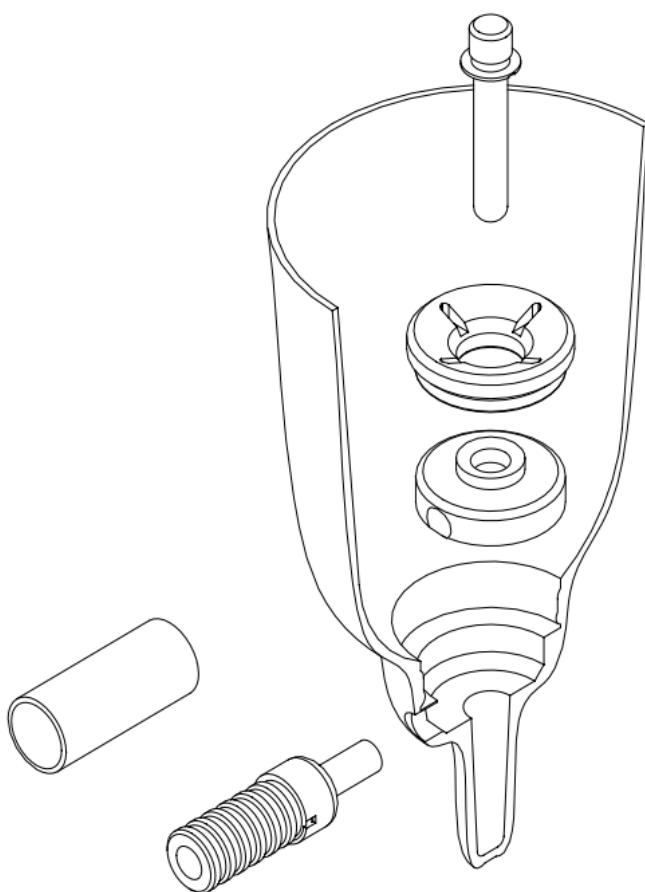

ENGLISH

TOOLS REQUIRED

8mm (5/16) socket wrench.

INSTALLATION

- Place Smooth Lock (L-692505) into socket, align release unit hole with hole in socket.
- Screw Release Unit (L-692501) in firmly and completely through hole on socket into Smooth Lock. Ensure that sufficient material is removed around the release hole to allow the Release Unit to be fully screwed in position without restriction in any way from the socket wall. Restriction of the Release Unit may alter the alignment and function of the Smooth Lock.
- Trim Push Button (L-693502) to length and replace Dust cap (L-693503). Trim the length of the protective sleeve (L-693504) accordingly, to fit the cosmesis.
- Screw Funnel (L-692110) into threads above Smooth Lock and torque to 8Nm (6ft.lb) using Cross Driver (item (L-692017) from Fabrication Tooling kit) and 8mm torque wrench.

NOTE: Resistance to removal of the pin may be experienced if the lock is not located correctly in the recess or if the funnel is not sufficiently tightened.

ATTACHMENT PIN

Apply Loctite 410 or 411 onto threads of Attachment Pin Smooth (L-693507) screw into Locking Liner.

Torque Attachment Pin to 4Nm (3ft.lb) with 7 mm wrench.

NOTE: Use only Attachment Pin Smooth. The locking pin should be exchanged at least once a year.

Limited one-year warranty.

Rated up to high activity level (K3).

All locks are fully interchangeable without necessitating fabrication of a new socket.

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EN – Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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Össur Iberia S.A.